



North Coast

Easy Grasp Rivet Gun

NC73510



Indications:

For adding straps to splints or other hard materials.

Instructions for use:

1. Open the handle of the rivet to its widest position.

2. Insert the sharp end of the rivet into the small hole at the head of the rivet gun.

Note: The collar of the rivet must rest against the opening.

3. Place the rivet in the desired location for attachment.

4. Squeeze the handle together. Repeat (if needed) until the rivet pops free and the sharp nail end is cut off.

5. Open the handle to its widest position. Shake the nail out.



Items needed for use:

Rivets and Plates (100)

(Sold Separately)

NC12723 $\frac{1}{8}$ " to $\frac{3}{8}$ " (3.2 to 9.5 mm)

NC12724 $\frac{1}{4}$ " to $\frac{1}{2}$ " (6.4 to 12.7 mm)

NC12725 $\frac{3}{8}$ " to $\frac{5}{8}$ " (9.5 to 15.9 mm)