# SplintCushion™

## Intended Purposes

A non-sterile adhesive backed foam material used to support, soften or line/pad equipment surfaces, including splints.

**Indications** A barrier against abrasion, used to improve fit.

**Contra Indications:** Skin irritation, allergy to PVC/NBR foam, or fluctuating edema.

## Instructions for Use / Proper Fit

Measure and cut padding to the desired length. Peel adhesive backing from the cut size and apply the sticky back to the target surface, aligning the top edge of both surface & adhesive. If bubble forms, compress or unpeel for reapplication. Avoid direct contact with open wounds. **Stretch** model can be heated in water and molded to customize padding.\*

**Warning:** If skin becomes irritated, seek advice from a healthcare professional.

**Instructions for Care** Spot clean with a damp sponge and air dry.

*\*Optional: water heat for a custom fit to surface*  
Examples: application to thermoplastic or other water/heat resistant surface. Apply the cut piece to surface and place in water bath. Mold or stretch to desired shape. Cool before application to skin.

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