Goodie-Strap™

Intended Purposes Non-sterile device used to assist with holding objects during daily activities such as eating utensils, hair brush or comb with handle.

Indications Persons with limited finger range of motion, dexterity or weak grasp.

Contra Indications Skin irritation, allergy to polyester or spandex.

Instructions for Use / Proper Fit

Before use, identify the smaller versus larger loop ends. Slide smaller loop over the utensil end (or other object) and onto the object handle. Pull the larger loop over the opposite end of the object. Strap should lay flat and untwisted against handle. To use, slide four fingers between the strap and handle. Adjust for position and comfort.

Note: D-ring on end of strap may aid to secure or loosen the strap for those users with limited finger strength or dexterity.

Warning: If strap feels too tight, skin becomes irritated, or if pain, stiffness, or numbness occur, discontinue use and seek advice from a healthcare professional.

Instructions for Care Hand wash in cool water and mild soap. Air dry. Do not place in dishwasher.

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North Coast Medical, Inc.
Morgan Hill, CA 95037–U.S.A.
ICON (LR) Limited
Allenspach Medical AG