Norco® Universal Quad Cuff

Intended Purposes Non-sterile device with cylindrical foam, used to assist gripping with the hand during daily activities such as eating with utensils, using a toothbrush, writing with a pen, or hair brushing.

Indications Persons with limited finger range of motion, dexterity or weak grasp.

Contra Indications Skin irritation, allergy to nylon or materials made from rubber.

Instructions for Use / Proper Fit

Unfasten or loosen elastic strap from around cylindrical foam. Slide object between elastic cuff and cylindrical foam. Fasten elastic strap, adjusting tension as desired. Loosen d-ring strap and slip hand in, so the cylindrical foam fits comfortably in the palm. Secure the d-ring strap by wrapping the hook section across the top of the hand and attach to loop. Fit should be comfortable. If tight, loosen strap.

Note: D-ring on end of strap may aid to secure or loosen the strap for those users with limited finger strength or dexterity.

Warning: If cuff feels too tight, skin becomes irritated, or if pain, stiffness, or numbness occur, discontinue use and seek advice from a healthcare professional.

Instructions for Care Hand wash in cool water and mild soap. Air dry. Do not place in dishwasher.

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North Coast Medical, Inc.
Morgan Hill, CA 95037—U.S.A.
ICON (LR) Limited
Allenspach Medical AG