

## 3pp® Soft Finger Loops and Wraps



### 3pp® Buddy Loops®



Non-slip, foam-lined 3pp® Buddy Loops® are the most effective way to secure fingers to protect or assist motion for **jammed or sprained fingers**.

### Description

1/2" (1.27 cm), 3/4" (1.9 cm) & 1" (2.5 cm)  
All widths are 5" (12.7 cm) long  
Pkg of 5, 25 & 100

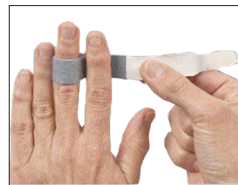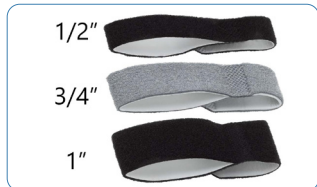

### 3pp® Final Flexion Wrap™



1 size fits most

The adjustable 3pp® Final Flexion Wrap™ makes it easy to **restore end range DIP and PIP joint flexion**. Static progressive design puts the user in control to regain full finger flexion.

### Description

Length 6 3/4" (4.45 cm)  
Width 1 1/4" - 5/8" (3.17 - 1.6 cm)  
Pkg of 5 & 25



## FAQ's

### 3pp® Buddy Loops®

What is the difference between the 3 sizes?

- 1/2" is recommended for smaller hands and for use during light activities
- 3/4" is a good all around size and can be trimmed if necessary
- 1" can be used on large hands and during sports

### 3pp® Final Flexion Wrap™ (FFW)

How does this wrap work?

- The FFW applies a static progressive force on the DIP and PIP joints to gradually increase IP flexion
- Wearer can adjust the wrap to tolerance for optimal results and compliance
- Wearing time is 5 to 10 mins. or as recommended by doctor or therapist