Norco® Dema Wrap®

Intended Purposes Provides light support and compression to strains and sprains. It also works well for controlling edema. Helps temporarily provide support. May be used after injury to help reduce swelling.

Indications Sports injuries, strained muscles, and joints, swelling, other minor hand/joint conditions.

Contraindications Skin irritation, allergy to polypropylene, natural rubber (standard model), polyisoprene latex (latex free model), loss of circulation.

Instructions for Use / Proper Fit

Wrap the bandage around the affected area, overlapping layers using the spiral technique (approximately two to three layers). Cut to length. Wrap with light stretch to provide a light compression. Avoid direct contact with open wounds.

The wrap should be comfortable to wear and not restrict circulation. Loosen wrap if compression feels too tight.

Note: Note: Wrap will adhere to itself. Secure wrap ends with tape for a more permanent hold.

Warning: If compression feels too tight, loosen wrap. If body part distal to the wrap changes color, there is an increase in swelling, skin becomes irritated, or if pain, stiffness, or numbness increases, discontinue use and seek advice from a healthcare professional.

Instructions for Care Spot clean with cool water and mild soap. Air dry.

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North Coast Medical, Inc.
Morgan Hill, CA 95037–U.S.A.
ICON (LR) Limited
Allenspach Medical AG