Intended Purposes

The device is used to safely and evenly divide pills for an equal amount of ingredients in each piece, by cutting or by crushing pills into powder. Whole, cut, or crushed pills can also be stored in the unit.

Indications Facilitate swallowing of medications.

Contra Indications Improper dosage from bad split, cross-contamination of different drugs, allergic reaction from residue contamination.

Caution: The top chamber includes a built-in safety shield which hides the sharp blade when the lid is opened.

Instructions for Use

Splitting Pills: Open top of splitter. Position pill in the cutting area so it rests centrally flush against the upper chamber. Lower top and close firmly to split pill. If the pill is oblong, cut across the pill, not longitudinally.

Crushing Pills: Unscrew the top of crushing unit and place desired pill dose into the crushing chamber. Replace the top on and twist until pills are crushed into a fine powder. As necessary, store ground pill in the chamber.

Warning: Pill splitter utilizes a blade. Do not place fingers on blade or in compression area to avoid injury. Keep Pill Pal out of the reach of children.

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