Performance & Value

Dema Grip®

Dema Grip® Gentle

Intended Purpose

Dema Grip®: Provides moderate to heavy compression and support for the management of strains, sprains and swelling.

Dema Grip® Gentle: Provides light to moderate compression and support for the management of strains, sprains and swelling.

Indications

Use for strains, sprains, soft tissue injuries, general edema and post-burn scarring.

Contraindications

Avoid use for patients with arterial insufficiency, ischemia, uncontrolled congestive heart failure, acute dermatitis, weeping dermatosis, and cutaneous sepsis.

Instructions for Use

1. Measure circumference of the largest area needing compression.
2. Divide measurement by 3.
3. Select size closest to measurement.
4. Cut piece twice the length of the area.
5. Apply one layer of the bandage over the area, then fold remaining half back over to form a double layer of compression.
6. Top layer edge should extend about 1.5" (3.8 cm) past bottom layer edge.

Warning

If numbness or discomfort occurs or swelling increases, discontinue use and seek medical advice.

Instructions for Care

Machine wash in warm water without bleach. Air dry.

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North Coast Medical, Inc.