

**Intended Purposes**

Non-sterile, device with soft terrycloth cover used to assist in pulling on socks or stockings. Shape trough with the use of one hand.

**Indications**

Persons with difficulty bending over, dizziness, or vertigo.

**Contraindications**

Limited hand strength. Allergy to polypropylene or polyester.

**Instructions for Use**



1) Gather entire sock over end of sock aid.



2) Hold strap and drop sock aid onto floor.



3) Slide foot into sock aid opening.



4) Pull upward on strap to bring sock over heel and up leg.

**Warning:**

Keep out of reach of children. Plastic bag and small parts may pose a choking hazard.

**Instructions for Care**

Hand wash in cool water and mild soap. Air dry.