




# HAUSMANN INDUSTRIES

130 Union St • Northvale NJ 07647 USA • Tel: (201) 767-0255 • Fax: (201) 767-1369 • [www.hausmann.com](http://www.hausmann.com)

## INSTALLATION INSTRUCTIONS

### Model 1671 – PORTABLE SINGLE MIRROR

**TOOLS NEEDED**  
Phillips Screwdriver

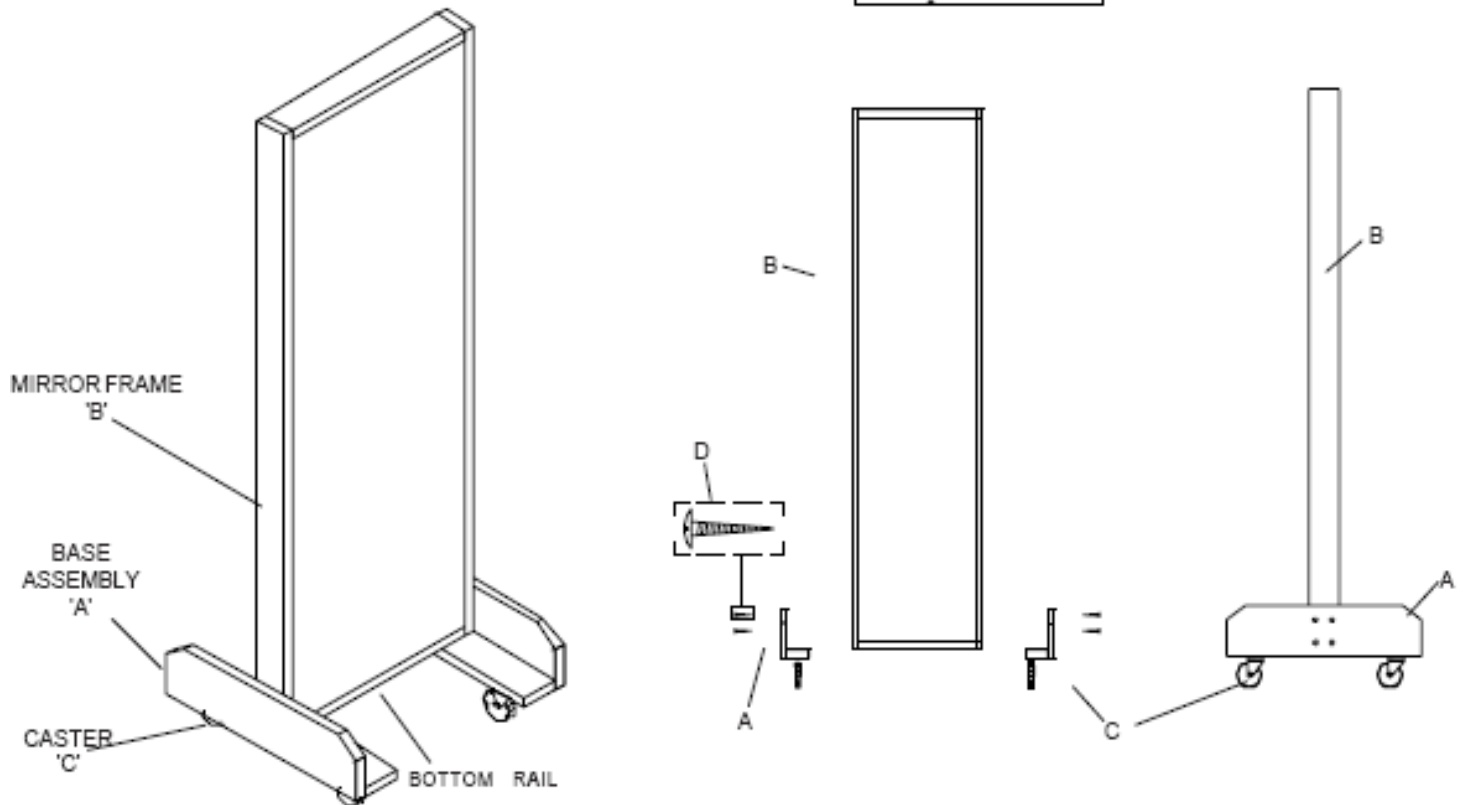

1. Place Mirror Frame (B) on edge of table, slip Base Assembly (A) onto mirror frame bottom.
  2. Secure Base Assembly (A) to Mirror Frame (B) with four (4) 1 3/4" #8 Truss Head Screws (D) provided.
  3. Turn Mirror Frame (B) over and repeat step 2 for Base Assembly (A) on the other side.
- \* Casters (C) come pre-attached to Base Assemblies (A).

*Note: For ease of assembly, apply wax or soap to screws.*

UNIT IS SHIPPED IN (1) CARTON

#### Parts List

PART:	PART #:	QUANTITY:	DESCRIPTION:
A	1671-001	2	Base Assembly
B	1671-002	1	Mirror Frame, with Laminate back and Shatter-Proof safety backing
C	1671-003	4	2" Diameter Rubber Casters
D	1671-004	8	1 3/4" #8 Truss Head Screws

**Warranty** Hausmann Industries, Inc. warrants each Hausmann product to be free of defects in material and workmanship, and agrees to remedy any such defect or to furnish a new equal part in exchange therefore. This warranty covers defects in materials and workmanship for three years from date of original purchase, provided such product is used for the normal purpose for which intended and damage is not the result of abuse or misuse.



Please join Hausmann's effort to reduce paper use and keep the planet green by using our paperless Warranty Registration form. Visit [www.hausmann.com](http://www.hausmann.com) and click on **Warranty Registration**. If you do not have web access or prefer the form be mailed or faxed to you, please call Hausmann Customer Service.