Norco® Big-Grip™ Telescoping Inspection Mirror

Intended Purposes

A stainless-steel extendable mirror with non-slip handle grip and optional lights used to facilitate the inspection of hard-to-see or obstructed areas.

Indications Increased viewing range.

Contra Indications: Allergy to Acrylonitrile Butadiene Styrene (ABS) thermoplastic or stainless steel.

Instructions for Use / Proper Fit

Unfold the inspection mirror and remove the protective film. Slide power switch on back of mirror to the "On" position for illumination. Adjust lanyard for best fit around wrist.

Straighten and extend the stainless steel retractable telescopic arm to the length necessary for viewing desired area.

For optimal visibility, adjust lighting by rotating LEDs (D1).

Troubleshooting If the light ceases to function, replace the battery (CR2032). With the mirror facing downward, slide the battery cover outward and remove old batteries. Check for debris and ensure the compartment is clean. Install two new batteries in correct polarity orientation (+/-) and replace sliding door. Do not expose to moisture or use in wet areas. If the unit comes in contact with water, remove batteries and allow compartment to dry. Extending or sliding the telescoping shaft with too much force may cause failure. Do not twist the flexible shaft too tightly as it may damage the unit.

Warning: Clean between uses and avoid contact with open wounds. To prevent accidents or injuries use product only for intended purpose.

Instructions for Care

Spot clean body with mild disinfectant and air dry. Glass cleaner can be used on the mirror. Avoid moisture around battery compartment and LED switch.

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