



ORFIT Industries

Vosveld 9a
2110 Wijnegem, Belgium

ORFILASTIC

INSTRUCTIONS FOR USE

A. GENERAL PRODUCT INFORMATION

ORFILASTIC is an elastic terrycloth lining material with a backing of self-adhesive open cell foam. It is used to improve air transmission of orthoses, external immobilization devices and rehabilitation aids and to ameliorate the comfort of wearing these products.

B. PRODUCT RANGE

Art. No.	Colour
35111	skin
35117	black

Width: 50 cm wide

Length: 1 m

Thickness: 3 mm

C. PRECAUTIONS BEFORE USE

- * Cut the pattern before removing the release paper from the back (direction has no importance).
- * ORFILASTIC is fixed on the inside of the splint, after the splint has been formed.
- * Remove the release paper and stick ORFILASTIC on the required area.
- * Proceed with removing the paper while sticking ORFILASTIC in the orthosis under slight pressure. The pressure must avoid the emerging of folds.
- * Make sure that ORFILASTIC does not stick back to back: this is irrevocable.

D. MAINTENANCE AND WASTE MANAGEMENT

- * ORFILASTIC is NOT waterproof and has to be replaced when necessary.
- * After use, ORFILASTIC can be disposed of with normal household waste without harming the environment.

E. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

The instructions were written in accordance with the European Directive 93/42/EEC for Medical Devices. It is prohibited to make alterations to this text without prior approval from ORFIT INDUSTRIES N.V.



ref.:31301 – version 5
last update: 20/05/2014 - revision date: 20/05/2016



Listed with

