COMPRESSION + HEAT

Thumb CMC Restriction Splint

**Ortho
Thermic™**

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Thumb CMC Restriction Splint

INTENDED PURPOSE

Provides support with full finger function with localized warmth to the thumb CMC joint for pain relief, stiffness, and inflammation.




INDICATIONS

Osteoarthritis and medial subluxation of the thumb CMC joint. Relieve thumb discomfort associated with CMC joint arthritis, instability, sprain, tendinitis, or repetitive motion injuries.

CONTRAINDICATIONS

Loss of circulation and skin irritation or allergy to nylon, neoprene, spandex, or silicon.

INSTRUCTIONS FOR USE / PROPER FIT

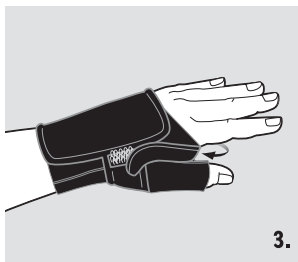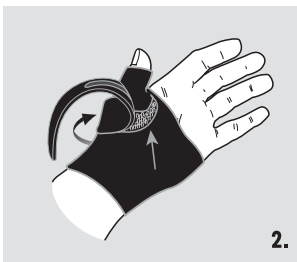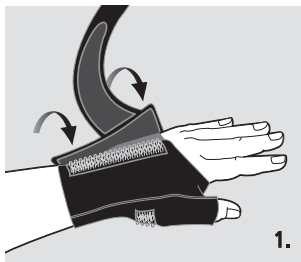1. Insert hand inside the splint with thumb sliding through thumb hole.
2. Pull the loop flap over the top of your hand and secure it to the hook area.
3. To adjust thumb support: Locate the three hook dots at the base of your thumb. Squeeze the outer dots to bend the metal insert for increased thumb support. Push on the center dot to reduce pressure.
4. Pull the strap around the base of your thumb without stretching it, fasten it to each of the three hook dots, and bring the strap through the thumb web-space, securing the end to the hook piece. **Note:** Size up if the strap must be pulled to fit or if the loop flap does not fully cover the hook area. Size down if the strap is too long to fit or if the loop flap fits loosely over the hook area.

WARNING

Discontinue use or adjust fit if patient shows signs of irritation, impaired circulation, increased pain or discomfort, or allergic reactions such as redness, itching, tingling, rash, or color changes in the affected area.

INSTRUCTIONS FOR CARE

Hand wash with mild soap in cool water. Air dry.

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		Size	MP Circumference	
NC79670	NC79671	S	6" to 7"	(15 to 18 cm)
NC79672	NC79673	S+	6½" to 7¼"	(17 to 18 cm)
NC79674	NC79675	M	7" to 8"	(18 to 20 cm)
NC79676	NC79677	M+	7¼" to 8¼"	(18 to 21 cm)
NC79678	NC79679	L	8" to 9"	(20 to 23 cm)
NC79680	NC79681	L+	8¼" to 9½"	(21 to 24 cm)
NC79682	NC79683	XL	9" to 10"	(23 to 25 cm)

CALIFORNIA PROPOSITION 65

WARNING

The foam adhesive layer may contain Toluene, which is known to the state of California to cause reproductive toxicity.

For more information, go to: www.p65warnings.ca.gov

North Coast Medical, Inc.
135 E Main Ave, Suite 110
Morgan Hill, CA 95037 – U.S.A.


ICON (LR) Limited
South County Business Park
Leopardstown, Dublin 18
D18 X5R3, Ireland

ICON (LR) Limited
500 South Oak Way
Green Park, Reading
RG2 6AD, United Kingdom

Allenspach Medical AG
Dornacherstrasse 5
4710 Balsthal, Switzerland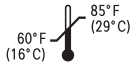

North Coast Medical, Inc.
www.ncmedical.com