## Rainbow™ FlexGel Hand Exerciser

### Intended Purposes

A temperature durable thermoplastic rubber ball with resistant levels ranging from light to heavy with warming and/or cooling capabilities. Used for grip strength training, dexterity improvement, and motor skills refinement of the hand and rehabilitation of the fingers, hands, and wrists.

**Indications** Reduce edema and help to improve range-of-motion (ROM) and increase finger dexterity.


**Contra Indications** Hand or finger weakness, or allergy to polyurethane.

### Instructions for Use / Proper Fit

• *For heat therapy use:* Microwave in 10 seconds intervals, until warm to the touch. Do not exceed 40 seconds total heating.

• *For cold therapy use:* Chill in refrigerator prior to treatment.

### Exercise Examples

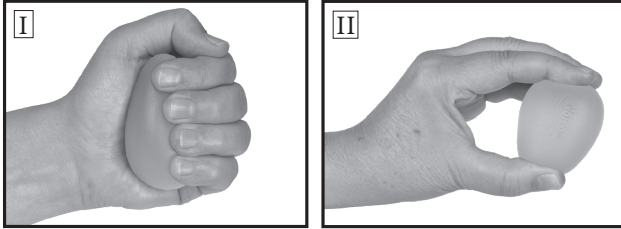\* *To progressively increase hand or digit strength, transition to the next resistance level between repeat intervals.*

**For hand strengthening:** With four fingers, grip and squeeze the exerciser, release and repeat (I).

**For targeted digit strengthening:** With the thumb and targeted finger, grip and squeeze the exerciser, release and repeat (II).

**Warning** If skin becomes irritated, soreness or inflammation increases, or user experiences pain, discontinue use and seek advice from healthcare provider. Avoid direct contact with open wounds.

### Instructions for Care

Rinse in warm, soapy water and pat dry with soft cloth. Paper towels are not recommended as they may leave lint on the product.

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North Coast Medical, Inc.
Morgan Hill, CA 95037–U.S.A.
ICON (LR) Limited
Allenspach Medical AG