



Ideal Medical Products
1287 County Road 623
Broseley MO 63932
1(800)321-5490

Ideal Rebounder Preventive Maintenance Guide

The Ideal Medical Products line of rebounders require little maintenance. This list covers the items to check:

- The springs and loops for defects.
- Cleaning with Formula 409 or equivalent cleaner.
- All bolt and nut connections making sure locknuts haven't backed off.

Preventive maintenance should be done every six months, or every three months under extreme use conditions, or using at max weight capacity for extended periods of time.

Discontinue use immediately if any part of the rebounder has been damaged, or if the unit has been modified in any way.

Spare parts are available through Ideal Medical Products. Only replace damaged items with parts from Ideal Medical Products.