




# Foot Screening Mapping Examples

## Touch-Test™ Sensory Evaluators


Key	Monofilament Size	Representation	Dorsal Surface Threshold	Plantar Surface Threshold	
	Callus	2.83	Green	Normal	Normal
	Pre-ulcer	3.61	Blue	Diminished light touch	Normal
	Ulcer	4.31	Purple	Diminished protective sensation	Diminished light touch
		4.56	Red	Loss of protective sensation	Diminished protective sensation
		5.07	Red	Loss of protective sensation	Loss of protective sensation
		6.65	Red	Deep pressure sensation only	Deep pressure sensation only

### Initial Evaluation - Visit #1

RIGHT FOOT: Superficial ulcer on plantar surface over the second metatarsal head.

LEFT FOOT: Pre-ulcer proximal to the first dorsal web space.


Patient education, treatment intervention and wound care management initiated.

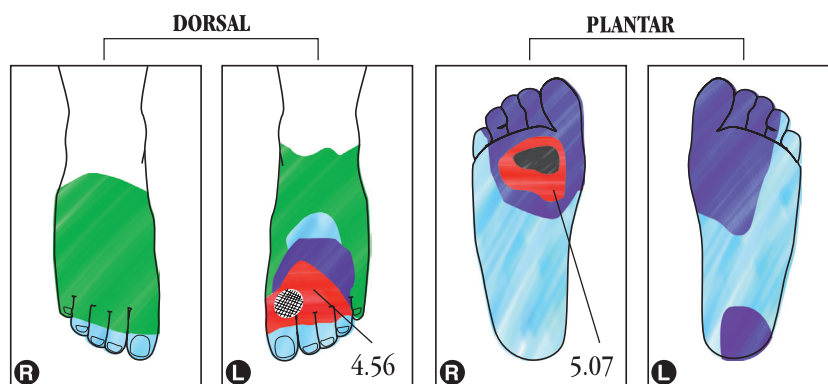

### Re-evaluation - Visit #2

RIGHT FOOT: Ulcer healed. Improved to diminished protective sensation on plantar surface over the second metatarsal head.


LEFT FOOT: Pre-ulcer healed. Loss of protective sensation proximal to the first dorsal web space.

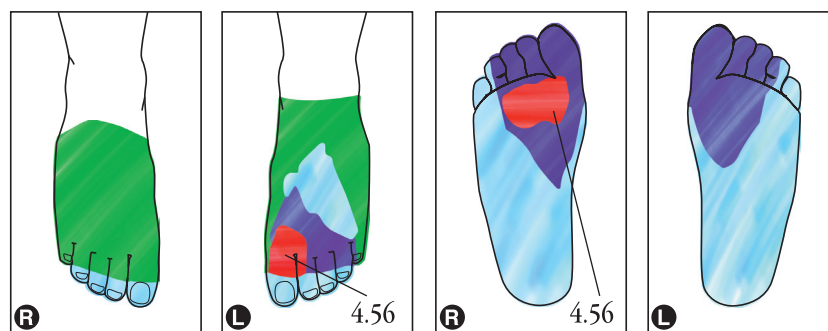

### Re-evaluation - Visit #3

BOTH FEET: Diminished light touch sensation at toes and plantar surfaces.

LEFT FOOT: Improved to diminished protective sensation proximal to the first dorsal web space.

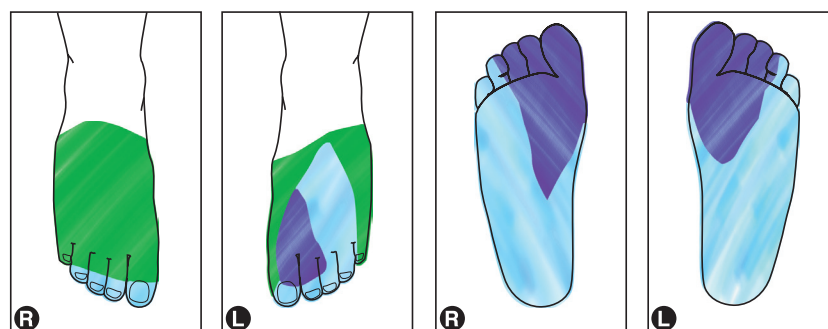

### Re-evaluation - Visit #4

RIGHT FOOT: Normal throughout.

LEFT FOOT: Improved to diminished light touch sensation over dorsal web spaces.

