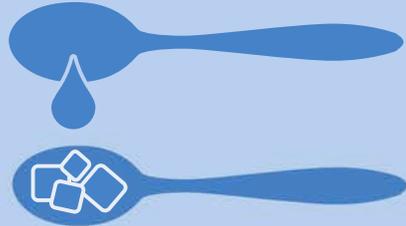
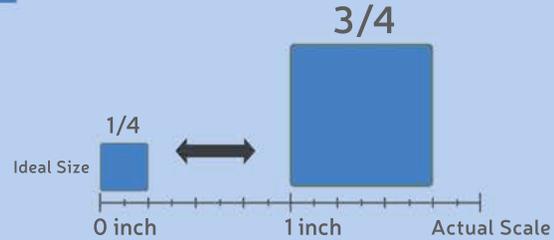
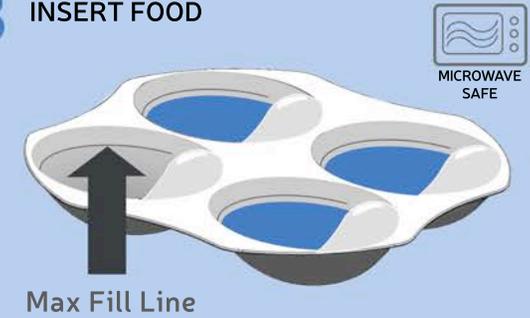


1 SELECT FOOD


Liquids: soups, cereals, beverages
Semi-solids: oatmeals, mashed potatoes, puddings
Solids: fruits, vegetables, meats

2 PREPARE FOOD


Obi works best when food is prepared within the range above.

3 INSERT FOOD


Fill bowls with food. Overfilling is not recommended.

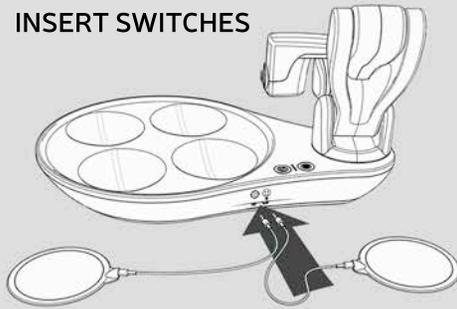4 INSERT SWITCHES


Place the device on a flat surface. Move the Arm to the right, as shown. Insert the desired Accessibility Switches.

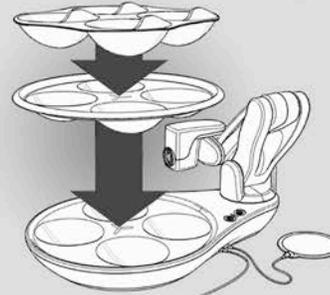5 INSERT PLATE & PLACEMAT


Insert the colored Placemat, then place the Plate on top.

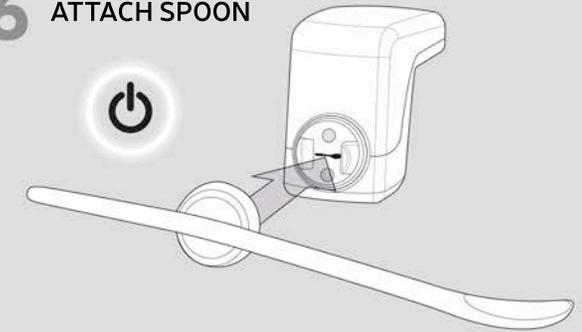6 ATTACH SPOON

Attach the Spoon and press the Power Button.

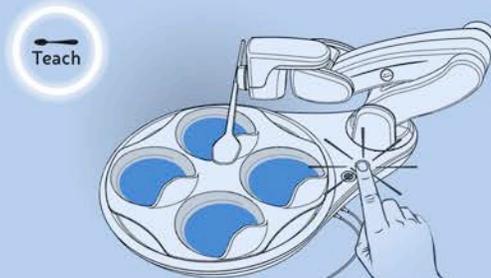
7 ENTER TEACH MODE

Press the Teach Button.

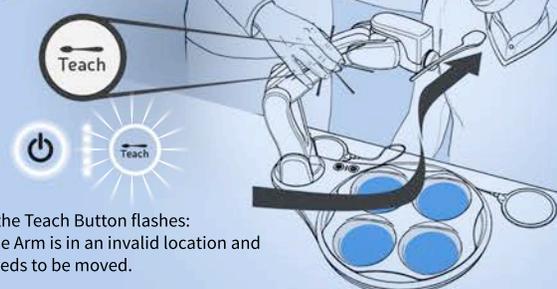8 MOVE ARM


If the Teach Button flashes:
The Arm is in an invalid location and needs to be moved.

Grasp as shown with a finger covering the Teach Icon. Move the Arm to a position in front of the user's mouth.

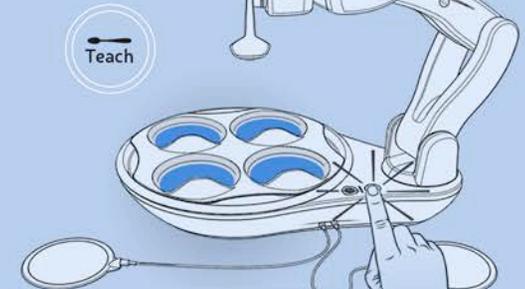
9 EXIT TEACH MODE

Press the Teach Button again to store the position.


EATING

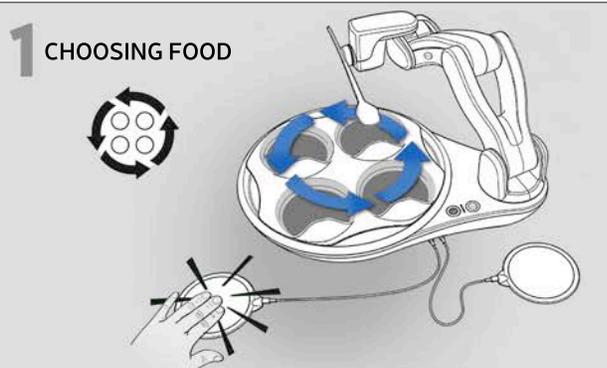

1 CHOOSING FOOD
The Choose Switch will move the Arm one bowl each time it is activated.

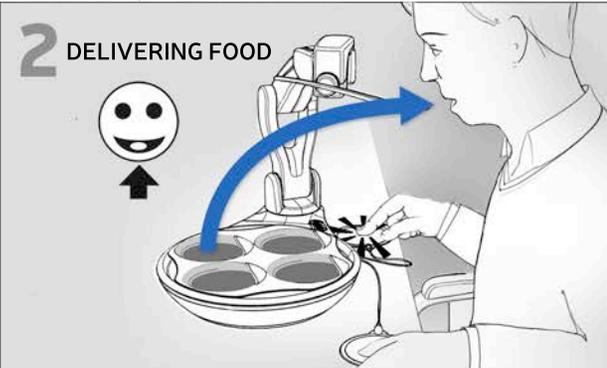


2 DELIVERING FOOD
The Deliver Switch will capture and transport food to the user's mouth.

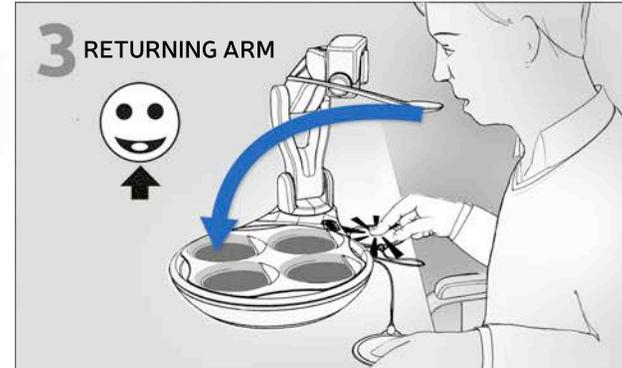


3 RETURNING ARM
Once the user removes the food from the Spoon, touch the Deliver Switch to return the Spoon to the same bowl.

CLEANING

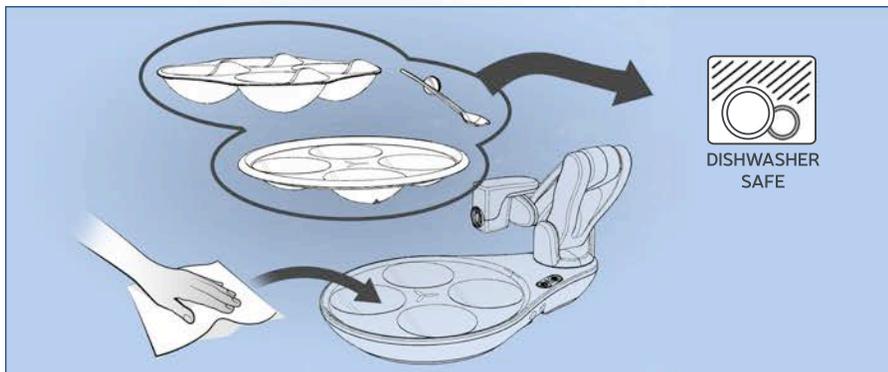


The Plate, Placemat, and Spoon can be cleaned in a sink or dishwasher (top rack). Only wipe down Obi's Base and Arm with a disinfecting wipe or damp cloth.

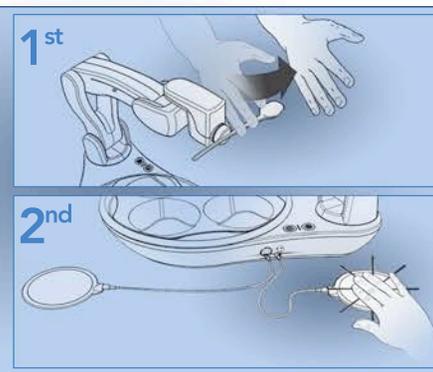


If the Power Button flashes, please remove the obstructing object. Once removed, touch either switch and the Arm will resume its motion.

COLLISION

CHARGING

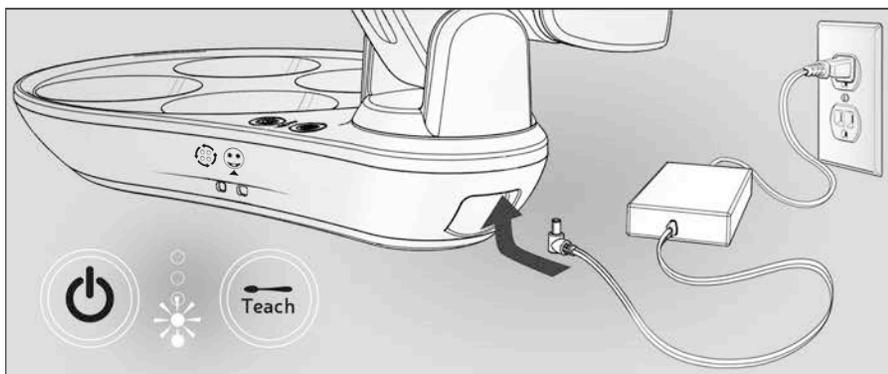

Connect the Charging Cable to Obi's Charge Port and an electrical outlet. All Battery Indicator Lights will illuminate when the device is fully charged.

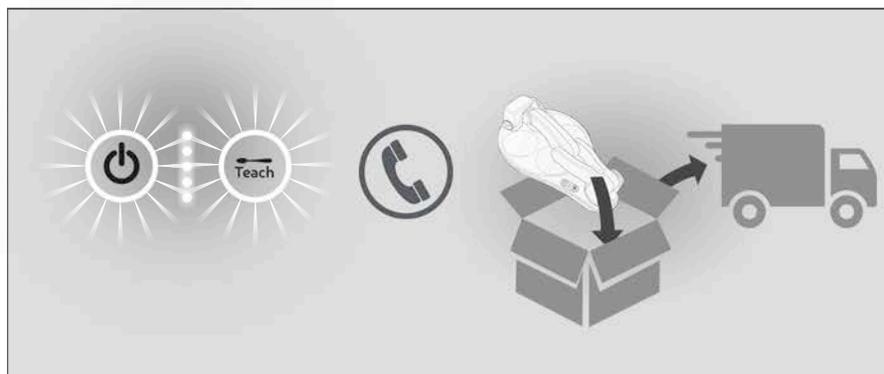

If all indicators are flashing, please power off for ten minutes. Afterwards, turn on Obi and if the problem persists, please call 1-844-HelpObi. If the issue cannot be resolved by phone, Obi authorized personnel will coordinate servicing.

SERVICE
REQUIRED