



North Coast

Bottle Warmer

Indications

Use to warm gels and lotions for more comfortable application when performing massage or using physical agent modalities.

Instructions For Use

To Mount:

Set the Bottle Warmer on a table or mount it to a wall using the keyhole slots on the back cover. Plug the power cord into a grounded 120 Volt outlet (GFCI preferred).

To Operate:

Flip the switch on the lower front panel to begin heating. All of the components are rated for continuous duty. However, turn the Warmer off overnight and on weekends because some gels and lotions tend to break down when exposed to heat for extended periods. Place gel, lotion or oil in the dispenser bottle provided and place the filled bottle back into the sleeve (bottle receptacle). On first turning on or filling the bottles, it may take up to 45 minutes before the inside bottle temperature becomes warm to the touch. The thermostat is adjustable from 90° to 120° F. Turn the knob on the side of the Warmer to suit individual needs.

To Clean:

UNPLUG THE BOTTLE WARMER BEFORE CLEANING!

Avoid getting any liquids into the empty receptacles. The Warmer frame is powder-coated white for easy cleaning with a damp cloth.



Bottle Warmer

	# of Bottles	Dimensions
NC70492-1	1	6" x 5" x 5" (15 x 13 x 13 cm)
NC70492-2	2	6" x 8" x 5" (15 x 20 x 13 cm)
NC70492-3	3	6" x 11" x 5" (15 x 28 x 13 cm)
NC70492-4	4	6" x 15" x 5" (15 x 38 x 13 cm)

Amperage Ratings @ 120 Volts 60 Hz.

NC70492-1	1.1
NC70492-2	2.2
NC70492-3	3.3
NC70492-4	4.4

Limited 1 Year Warranty



North Coast Medical, Inc.
Gilroy, California U.S.A.
Authorised Representative
Medica Surgical Innovations Ltd.
BB2 4PB UK