




North Coast

# Buddy Straps

## Indications

Buddy Straps promote use and increase range of motion of an injured finger by holding it firmly next to a finger that is functioning properly. They also may be used to prevent awkward or injurious work postures such as hyperextension of the MP joint. Made of lightweight hook and loop, Buddy Straps are hypoallergenic. One length fits all. May be trimmed with scissors for a proper fit. Buddy Straps are sold in a package of ten. Latex free.


## Instructions For Use

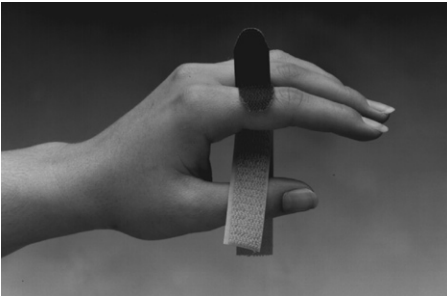

1. Hold cinched area of hook and loop strap between fingers.



2. Wrap one side of hook across finger. Bring up corresponding loop around finger and attach to hook.

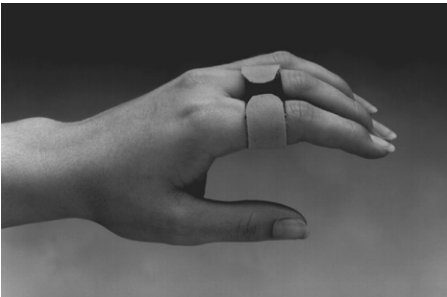

3. Adjust for a snug fit. Trim width or length as needed.
4. Repeat with other hook and loop straps.

## Instructions For Care

Wash in warm water with a mild soap. Air dry. Do not dry in a dryer.

### Width

NC15378	5/8" (1.6 cm)
NC15379	1" (2.5 cm)
NC15380	2" (5.1 cm)



North Coast Medical, Inc.  
Gilroy, California U.S.A.  
Authorised Representative  
Medica Surgical Innovations Ltd.  
BB2 4PB UK