



Button Hook/Zipper Pull

Introduction:

This special device makes managing buttons and zippers easier. Ideal for persons with limited finger dexterity or who have use of only one hand.

Button Hook

Instructions for Buttoning:

Fig. A

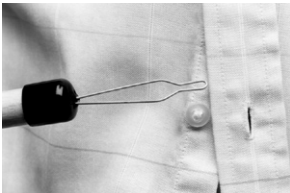


Fig. B


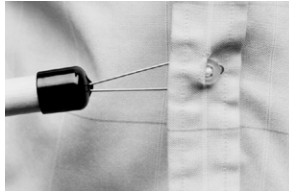


Fig. C

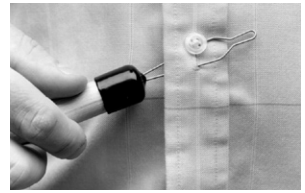

1. Slip the wire filament into the button hole. (Fig. A)
2. Hook the button with the wire filament. Then raise and turn the button to pull it through the button hole. If you need support on the shirt to keep it steady while you button, hold the shirt tails with your free hand. (Fig. B)
3. Pull the button through the button hole. Then slip the wire filament off the button. (Fig. C)

Instructions for Unbuttoning:

Fig. A

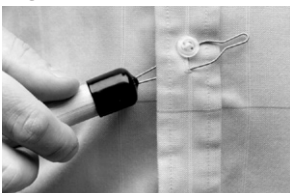


Fig. B


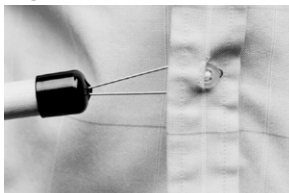


Fig. C

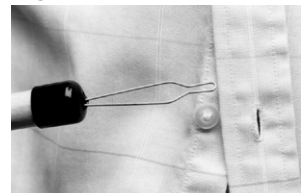

1. Slip the wire filament through the button hole and slip hook over the button. (Fig. A)
2. Twist and pull the button until it slips out of the button hole. If you need support on the shirt to keep it steady while you unbutton, hold the shirt tails with your free hand. (Fig. B)
3. Slip the wire filament off the button. (Fig. C)

Button Hook/Zipper Pull Instructions

Zipper Pull

Instructions for Use:


Fig. A



Fig. B



Fig. C

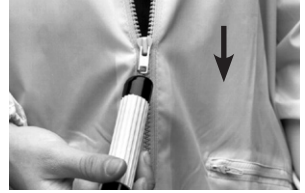

1. Insert the hook into the hole of the zipper tab. (Fig. A)
2. Using the other hand or arm, hold the end of the zipper as shown above to make sure the material doesn't get caught while you zip it. (Fig. B)
3. Pull up to open or down to close the zipper. (Fig. B and C)

Order Information

NC28616 Grip Handle Button Hook/Zipper Pull Combo

NC28635 Button Hook/Zipper Pull Combo

To be used under the guidance of a
qualified medical professional.



CE

North Coast Medical, Inc.
Gilroy, California U.S.A.
Authorised Representative
Medica Surgical Innovations Ltd.
BB2 4PB UK