



# Stick-On™ Finger Hooks

NC12357



## Indications

A great alternative to gluing hooks to nails. Designed for use in the treatment of contractures, burns, amputation, and tendon repairs of the fingers. Added padding helps protect sensitive nail beds and fingertips. Flexible, breathable material adheres securely while allowing the joint to move freely. Low hook height decreases accidental snags. Can be removed and reapplied up to 4 times. Can remain on fingertips all day without adjustment or maceration. Easily trim to fit any size finger.

## Instructions For Use

1. Remove paper backing. Apply over nail of finger, just above DIP joint. Press firmly along proximal base of finger hook.
2. Fold sections around fingertip in steps 1, 2, 3 (see figure 2) as shown.

## Instructions For Care

Avoid lotions or oils, as they can reduce adherence. Occasional washing of hands will not cause damage to the fabric adhesive properties.



Figure 1

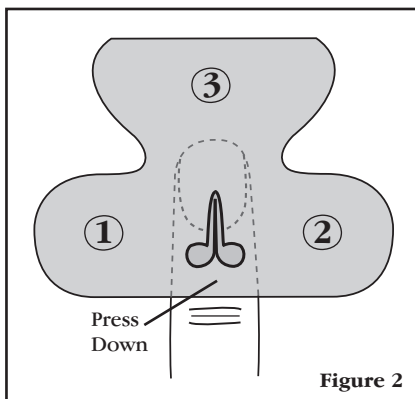


Figure 2

Latex-Free

To be used under the guidance of a qualified medical professional.

