


WINSFORD FEEDER[®]

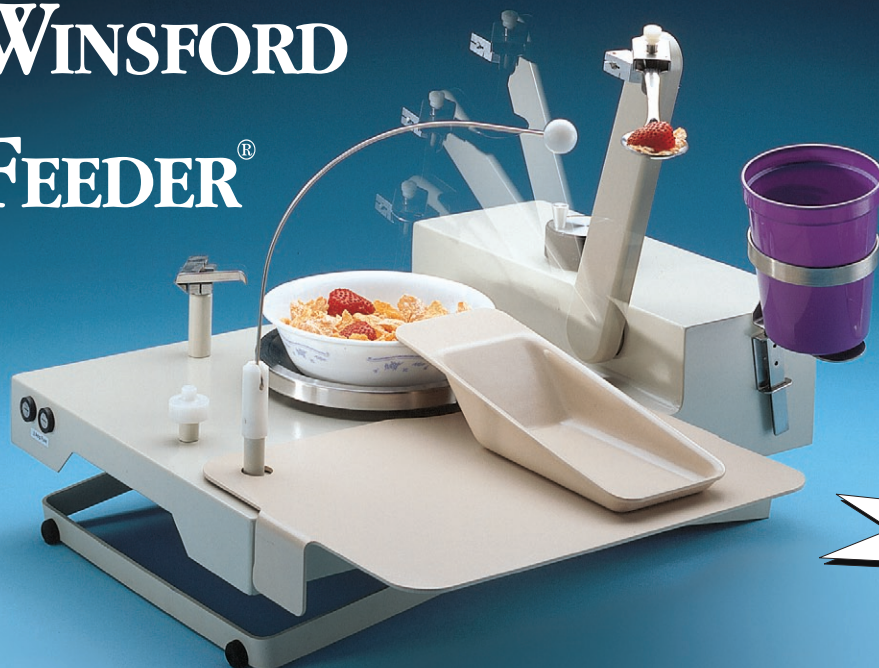

30-Day
Money-Back
Guarantee!

Bring independence back to mealtimes.

Eat soup, cereal and other food small enough to fit on a teaspoon.

WINSFORD FEEDER[®]

People who have lost the ability to feed themselves will truly appreciate this remarkable device. All that's required to operate the Winsford Feeder[®] is a slight move of the head. The Winsford Feeder[®] can also be controlled with other parts of the body such as hands, shoulders, hips or knees using the rocker switch.



Rocker switch, battery charger and instruction booklet are included. Using the charger during operation keeps the battery charged. The feeder also works without the battery.



The pusher moves the food onto the spoon once the switch is released.

The Winsford Feeder[®] includes a carrying case, battery, charger, Corelle[®] dinner plate, bowl, spoon, pusher, chin switch rod, stationary glass holder and turntable. Also includes a shelf and drip pan. Feeder measures 12" x 18" (51 x 46cm) and weighs 18 lbs. (8.2kg). An extra rechargeable battery is available separately. Instructions included. A free informational video is yours for the asking.

Independent eating success with the Winsford Feeder®

The plate and bowl rotate for easy accessibility.

The person eating sets the pace for the automated spoon.



“The Winsford Feeder® is terrific! It allows the students to be independent at mealtime while eating lunch with their friends.”


Leslie Anido
M.A. Sp. Ed.

The many innovative features make the Winsford Feeder® an ideal aid for more independent living:


- Operate with either the chin switch or the rocker switch.
- The person eating sets the pace for the automated spoon.
- The stationary glass or cup holder accommodates glasses or cups up to 3¼" (8.3cm) in diameter.
- The Winsford Feeder® includes a Corelle® dinner plate and bowl, spoon, pusher, chin switch rod, rocker switch, stationary glass holder, turntable, shelf, drip pan, and carrying case.
- Height adjustable.
- Can be used with liquids, such as soup or stews, and solid foods in pieces small enough to fit on a spoon.
- Powered by either AC current or the rechargeable battery.

Use with either the included dinner plate or bowl. To eat soup or cereal, remove the pusher and the plate. Attach the turntable, shelf, bowl and drip pan.

The height of the feeder is adjusted with the large knob. The Winsford Feeder® is designed for persons who are able to sit in an upright position. The feeder requires a minimum height of 10" (25cm) from the tabletop.

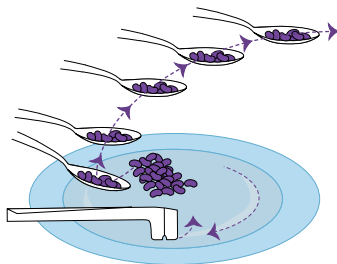

Upward automated spoon movement.

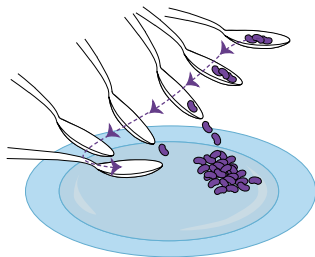

Downward automated spoon movement.



Pushing the chin switch to the left activates the pusher to move food onto the spoon and lifts the spoon up.



Pushing the chin switch to the left again causes the spoon to descend. It swings down into a vertical position so that its tip will meet the plate near the rim. The tip then slides along the plate, under the food, and stops at an angle to the plate.

If the spoon picks up more food than is desired, it may be returned to the plate. As it descends, the food will fall off onto the plate and the spoon will be refilled.



Pushing the chin switch to the right causes the plate to rotate so food is in position for the next bite. The rocker switch offers the same functions as the removable chin switch. Pressing one side of the rocker switch rotates the turntable and the other side activates the spoon.

NC38011	Winsford Feeder®
NC38010-2	Extra battery
NC38013	Video



North Coast Medical, Inc.
8100 Camino Arroyo • Gilroy, CA 95020
Local/Int'l: 408-776-5000 • Toll-Free: 800-821-9319
Toll-Free Fax: 877-213-9300 • www.ncmedical.com