



TORBOT GROUP INC.

Jobskin Division

JOBSKIN*
INTERIM CARE

Standard	w/Velcro Fasteners	Size/Length
NC53061	NC53071	SR = Small/Regular
NC53062	NC53072	SL = Small/Long
NC53063	NC53073	MR = Medium/Regular
NC53064	NC53074	ML = Medium/Long
NC53065	NC53075	LR = Large/Regular
NC53066	NC53076	LL = Large/Long

Measuring Information

1. Select RIGHT or LEFT hand measuring template.
2. Center pencil (or similar post) at point (P) to act as a stop. Place patient's hand, palm down, on the illustrated hand template and butt web space between middle and index fingers up against stop at (P).
3. Press patient's fingers together.
4. Trace horizontal line from tip of middle finger (T) to glove size diagram.
5. Trace a vertical line up from the base of little finger (B) to glove size diagram.
6. Find glove size at intersection of vertical and horizontal lines in glove size diagram.
7. Reference glove size and style to ordering information table part number.

