



## CMC Thumb Splint

### Order Information

#### IP Joint to distal wrist crease

|    |  |                 |
|----|--|-----------------|
| XS | 3 <sup>3</sup> / <sub>8</sub> " to 3 <sup>5</sup> / <sub>8</sub> " | (8.6 to 9.2 cm) |
| S  | 3 <sup>5</sup> / <sub>8</sub> to 4"                                | (9.2 to 10 cm)  |
| M  | 4" to 4 <sup>1</sup> / <sub>2</sub> "                              | (10 to 11 cm)   |
| L  | Over 4 <sup>1</sup> / <sub>2</sub> "                               | (over 10 cm)    |

#### **Firm Leatherette**

Left

Right

#### **Leather**

Left

Right

|    |         |         |           |           |
|----|---------|---------|-----------|-----------|
| XS | NC16330 | NC16331 | NC15515-7 | NC15515-8 |
| S  | NC16332 | NC16333 | NC15515-1 | NC15515-2 |
| M  | NC16334 | NC16335 | NC15515-3 | NC15515-4 |
| L  | NC16336 | NC16337 | NC15515-5 | NC15515-6 |


# Liberty™ CMC Thumb Splint

## Indications

The Liberty™ CMC Thumb Splint provides support and protection to the CMC and MP joints, while leaving the wrist and fingers free. The Liberty™ CMC Thumb Splint can be used for relieving symptoms associated with CMC and MP joint arthritis, thumb strain or sprain and repetitive motion injuries.

## Instructions for Use

1. Place hand over splint as shown (**Figure A**).
2. Fit splint to thumb, securing hook and loop closure around thumb (**Figure B**).
3. Thread the wrist strap through the slot (**Figure C**).
4. Pull the strap back over the wrist and secure with hook tab. The straps may be trimmed as needed (**Figure D**).

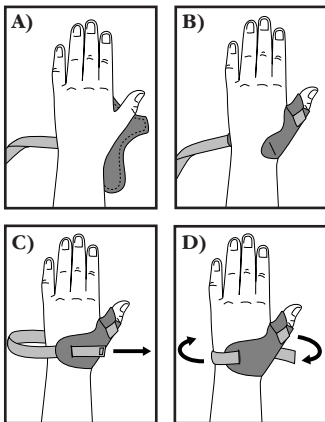

## Instructions For Care:

### *Leather*

Clean with a leather cleaner or wash with a mild soap in water. Air dry.

### *Firm Leatherette*

Machine wash and air dry; secure hook and loop prior to washing.

Single  
Patient Use



Manufactured by:  
North Coast Medical, Inc.  
www.ncmedical.com

©2013 North Coast Medical, Inc.  
Gilroy, CA 95020

REV15515pi\_0713